ZOOMLITE PRO

See clearer. Brighter. Smarter.

ENGLISH

ZoomLite Pro glasses provide **160% magnification** of objects without distortion, combined with the power of hands-free LED lighting. With a 120-degree field of view, everything appears brighter, sharper and more detailed.

WARNINGS:

- **DO NOT** use ZoomLite Pro as protective eyewear.
- **DO NOT** look directly at the sun while using ZoomLite Pro; this may cause immediate and irreversible damage to your eyes.
- **DO NOT** stare directly into the LED lights. This can cause vision problems.
- **DO NOT** wear ZoomLite Pro while driving.
- **DO NOT** drive while wearing ZoomLite Pro.
- **DO NOT** use alcohol-based cleaners to clean ZoomLite Pro lenses.

ATTENTION: For near vision use only. Ready-to-use glasses, no prescription required, but do not replace corrective lenses prescribed after an eye exam by a professional. Only regular eye exams can determine your vision needs and eye health.

CARE AND CLEANING: To remove dirt and smudges, use the included microfiber cloth or a cloth designed for optical lenses. For heavy dirt, use water and mild soap. **DO NOT** apply pressure while cleaning, as this could scratch the lenses.

INSTRUCTIONS FOR USE: ZoomLite Pro glasses are easy to use. They can be worn alone or over prescription glasses or sunglasses. Press the **ON/OFF BUTTON** located on each arm of the glasses to turn the lights on.

Use over other glasses: When wearing over prescription or sunglasses, adjust the distance to find the right visual point.

BATTERY REPLACEMENT:

↑
Switch: On/Off

* For battery replacement

Battery specifications:

4 batteries per pair of ZoomLite Pro glasses

Model: CR1220 button cell

Type: Li-Mn, <0.3g Li

Voltage: 3.0 V

Capacity: 38 mAh

Do not dispose of electrical products with household waste. Recycle them in appropriate bins and/or according to your local recycling guidelines.

Box contents:

- 2 LED glasses
(2 CR1220 batteries per arm, installed in the arms of the glasses)
- 1 pair of anti-blue light glasses
- 1 multifunctional cleaning spray
- 2 soft cleaning pouches
- 2 cords
- 6 additional CR1220 batteries

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